

IMPORTANT Read Both Sides in Full Before Using

AcquaBrite™ Take Home Whitening System with AcquaSeal™ Desensitizer

Instructions for Home Use:

For frequency and duration of treatment follow the instruction provided by your dentist. If no instructions were supplied use every other day for one hour at a time. Do not exceed a total of four applications unless recommended by your dentist. Each syringe will contain 2-4 applications depending upon amount of material used and size of tray. After initial bleaching touch-ups may be needed every 2-6 months.

1. Brush your teeth thoroughly before use.
2. If you generally have sensitivity use AcquaSeal Benz swab on your teeth before whitening. If not use when step 8 is reached.
3. Remove and save the cap. You will need it for storage.
4. Attach the red mixing tip to the whitening syringe. Push cap on and rotate to lock tip in position. Before applying to the dentist supplied tray push out enough material until it comes out amber, not clear.
5. Apply AcquaBrite through included mixing tip directly into dentist supplied custom tray. Apply AcquaBrite to the part of the tray that will contact the front of the teeth. Do not fill the tray. Use sufficient material to cover the inside front surface, this is where the front of your teeth will contact the mouth guard. If the guard overflows when you insert it, too much material was added.
6. Whitening is finished after 45 minutes. Remove tray at this time.
7. Rinse mouth thoroughly with water to remove AcquaBrite from mouth. Spit out the rinse water.
8. Desensitize using AcquaSeal Benz swab per instructions:
 - Hold swab vertical with striped end on the top
 - Snap gently the striped end; tip may break off, which is o.k.
 - Let AcquaSeal Benz flow into the swab tip on the bottom
 - Paint onto teeth that whitening was performed on and discard
 - Let air dry for 15 seconds
 - Rinse mouth with water.
9. Remove used mixing tip from AcquaBrite syringe. Twist to unlock and pull off. Throw away tip. Do not re-use old mixing tips.
10. Replace the cap that was removed from the syringe in step 3 back onto syringe. Twist to lock cap in closed position.
11. Refrigerate the sealed syringe. The syringe may be stored outside a refrigerator, in a cool dark place, with only a slight reduction in shelf life.

Additional Information:

AcquaBrite is a self-activating 16% carbamide peroxide bleaching gel.

Indications: Whitening of discolored teeth (vital and non-vital) caused by:

- Age
- Food-stains
- Medication (e.g. tetracycline)
- Fluorosis
- Non-vital teeth/Endodontically treated teeth
- Bleeding stains cause by pulpal trauma

Contraindications (Do NOT use if):

- Pregnant or lactating
- Patients known to be allergic to ingredients or similar products
- Untreated caries (cavities)

Warnings:

- Carbamide Peroxide-contact with skin, mucous or eyes will cause irritation, if this occurs discontinue use. In case of contact, rinse/flush with plenty of water for at least 1 minute and consult physician.
- Do not swallow.
- Keep out of reach of children

If sensitivity is noted during procedure and does not go away with use of the Desensitizing AcquaSeal Benz Swab discontinue use and contact your dentist. There may be other problems with your teeth.

Storage:

Refrigerate (2-6°C/35-43°F). Keep out of sunlight and/or extended periods of heat.

Made in Germany for AcquaMed Technologies, Inc., 33W480 Fabyan Parkway Suite 105 West Chicago IL 60185
USA Tech Support: (888)656-9620 or (630)232-9704