

IMPORTANT Desensitizer INSTRUCTIONS

- 1 Hold swab vertically with tip with black line on top.
- 2 Snap at line and allow liquid to drain into tip.
- 3 Holding lip away from teeth, apply to upper teeth by rubbing on the surface of the teeth.
- 4 Apply only to the teeth, not on gums or the lips.
- 5 Continue to hold lip away from surface for 15 seconds
- 6 Rinse thoroughly with water.
- 7 Repeat procedure on lower teeth holding lip away from teeth.
- 8 Place material on teeth only and rinse well when complete.

IF YOU HAVE ANY QUESTIONS OR IF IRRITATION OR SWELLING OCCURS ON LIP DISCONTINUE USE AND CONTACT YOUR DENTIST.